TAXIDEN[®]

Biochemical identification strips

SIMPLICITY ● inoculum simplified with Inoclic[®] : work starting from only one colony, very little handling, very simple use.

EASY READING ● change of color if tests are positive.

VISUAL MONITORING ● all the results can be controlled visually.

ECONOMIC ● no control of purity ; reduced size involving few waste.

FLEXIBILITY ● strips adapted to each type of organisms.


SPEED ● instantaneous reading with SirScan[®]Micro.

TRACEABILITY ● SirScan[®]Micro is the only system which ensures a complete traceability of results, the management of the batches and expiry dates.


Operating instructions TAXIDEN® strips

TAXIDEN® strips reading TAXIDEN®, identification motor

- **TAXIDEN® Software.** Entry of the results in the software, displaying/printing of the results.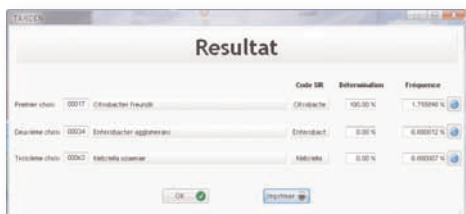

- **SirWeb® TAXIDEN® Software.** Entry of the results in the software, displaying/printing of the results.

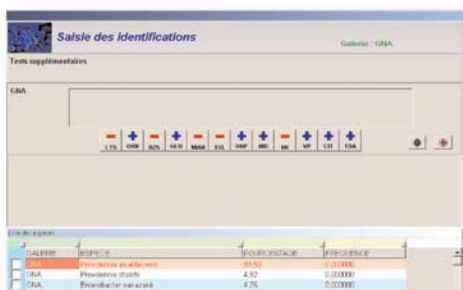
- **Instantaneous automatic reading by the reader SirScan® Micro+.** Automatic backup of all images.

Diagram of use for the TAXIDEN® identification strips


Realisation of inoculum

- ➔ With Inoclic® only one colony is sufficient (except streptococci : 2 Mac Farland)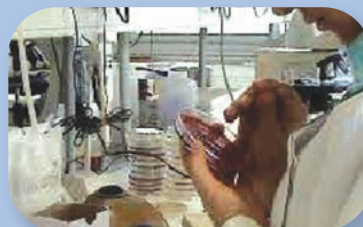

Inoculation of strips

- ➔ Using a pipette, deposit of 100 µl in each well (plus hippurate test for streptococci)
- ➔ Addition of paraffin oil in the black wells

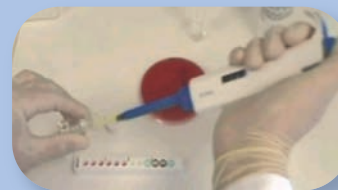Incubation and revelation

- ➔ Incubation during 18/24 hours at 35°C-37°C (except not fermenting, 48 hours)
- ➔ Addition of the revelation reagents in the specific wells

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